# DENTAL DIAMOND BURS USER MANUAL

## Model Information: Reference

[www.bosphorus.com.tr](http://www.bosphorus.com.tr)

Be sure to read the instruction before use.

Non Sterile

### Intended use

These products are made of stainless steel coated with small diamond crystals. It is a rotary cutting device designed to fit a dental handpiece and used to cut hard structures in the mouth. It can also be used to cut hard metals, plastics, porcelain and similar materials.

Produced for all kinds of carving and cutting operations which are used by dentists during medical treatments. Area of application These general instructions for use and safety recommendations apply to all products and have to be generally observed. Nonobservance of these instructions for use and safety recommendations increases the risk of injury and may impair the proper function of the instruments.

Separate locations of use are available on our web site

[www.bosphorus.com.tr](http://www.bosphorus.com.tr) for products that require more detailed information.

These take precedence over the general instructions.

### Storage

Instruments in their original packaging, away from light and heat in a clean and dry environment. Do not store in the same room as solvents or chemicals. Do not damage the packaging material. The product is subject to improvement/change without notice. For this reason, it is recommended to use the first purchased products first.

### Packaging

Packing in a box: 5 pcs

### Possible Combinations with the Dental Handpiece

Use a dental handpiece that will ensure full grip of the shaft. (Friction shaft shank diameter is 1.60mm, Contra Angle shaft diameter is 2.35mm.) Use a handpiece that can be controlled at the allowable engine speed.

### Contraindications and Prohibitions

Do not use this product for a patient with sensitization or allergic reaction.

For professional use only. Use it for its intended use.

### Indications

#### 1. Proper use

- The instruments have to be prepared prior to first use.
- Make sure that only technically and hygienically perfect and cleaned power systems (hand pieces and contra-angles) are used.
- Depending on their shank type, insert the instrument into the chuck as deeply as possible. Make sure that they are properly locked.
- The instrument must be rotating at the desired speed before contact is made with the tissue or material.
- Avoid jamming and using the instrument as a lever. Excessive contact pressure has to be avoided.
- For eye protection wear safety glasses. Use appropriate respiratory protection (mouth and nose). In the dental laboratory, use appropriate suction unit.
- Avoid contact with the instruments' working parts as this may increase the risk of injury.

Make sure to provide sufficient cooling by means of air/water spray. Additional external cooling is required when using instruments with extra-long shanks or oversized working parts. Damaged or corroded instruments have to be discarded.

## 2. Recommended speeds

HEAD DIAMETER(ISO ø (1/10 mm))	FG(Max. RPM)	HP(Max. RPM)
007-014	300,000	160,000
016-023	280,000	120,000
025-045	180,000	100,000
047-065	80,000	80,000
066-093	60,000	50,000

For optimum results when using Bosphorus branded burs, 50% of the maximum speed should be used..

## 3. Contact pressure

Excessive contact pressure (> 2N) has to be avoided.

- In cutting instruments, this can lead to damage to the working part and to chips of the diamond as well as an excessive generation of heat.

## 4. Number of times rotary instruments can be used and shelf life

Diamond dental burs: - Recommended Max 15 x  
The recommended shelf life of diamond dental burs is 10 years.

## 5. Disinfection, cleaning and sterilisation

Those instruments that are supplied non-sterile have to be prepared prior to first use.

### 5.1. Manual reprocessing

The instruments are to be disinfected with appropriate cleaning and disinfecting agents recommended for these products; remain 15 minutes in (ECOLAB- Sekudrill) cleaning and disinfection liquid 2-3 ml with 3ml water composition, make sure that the instruments do not come in contact with each other during this process.

### 5.2. Mechanical reprocessing

The instruments have to be treated with suitable detergents and disinfectants that are recommended for this purpose. Observe the instructions of use provided by the manufacturer.

Make sure that the instruments do not come in contact with each other during the cleaning in the ultrasonic bath. Inspect the clean instruments visually. Separate and discard damaged or blunt instruments. Thoroughly cleaned instruments are an essential condition for a successful sterilization.

### 5.3. Sterilization

Make sure that the instruments are sterilized according to a validated procedure suitable for the sterilization of medical products. Successful sterilization also depends on the type of product to be sterilized, the packaging and the loading set-up of the sterilization device. The operator of medical products is responsible for seeing that proper treatment is carried out by qualified personnel, using the appropriate materials and suited equipment, as recommended by the Commission for Hospital Hygiene and Infectious.

**Caution:** Operative burs must be sterilized **before each use** to avoid potential transfer of infectious organisms. Autoclaving: Dip the instruments in a corrosion inhibitor such as 1% sodium nitrite, prior to the sterilization cycle. In the unwrapped condition, the instrument can be sterilized by a pre-vacuum flash or gravity flash steam sterilization, utilizing a steam sterilizer. For the pre-vacuum Flash method, operate at full cycle with a dwell at 134°C minimum for 5 minutes.

## 6. Specific instructions for individual instrument types

- Avoid any contact with H2O2 (hydrogen peroxide). Make sure that the specified immersion times in the cleaning and disinfecting agents are not exceeded.

## 7. Potential effects of using worn instruments

- Discard any instruments with damaged or worn working parts as the use of damaged or worn instruments would have a negative effect on the work result.
- Damaged and deformed burs will cause the instrument to vibrate and lead to poor preparation margins and rough surfaces.
- Please avoid excessive contact pressure during use as this may result in damage to the working part (Nicks on the blades, premature blunting and excessive heat generation).
- To avoid undesirable heat generation during preparation, make sure to provide sufficient cooling by means of air/water spray (at least 50 ml/min).
- Additional external cooling is required when using instruments with a total length of more than 22 mm or a head diameter exceeding 2,5 mm.
- Not observing the maximum permissible speed will result in an increased risk of injury.
- There is an increased risk of infection in reusable instruments which have not been properly reprocessed.

## 8. Safety and possible side effects

The above mentioned recommendations with respect to cooling, contact pressure, disinfection, cleaning and sterilization are to be strictly observed. The instruments should only be used for the intended application, as per the symbolic identification. Non-observance of these safety recommendations may lead to damage of the power system and/or injury, such as thermal necrosis, undesired preparation of tissue, damage to tissue or nerves, violation of the biological width, or infections. During preparation, some instruments may generate metallic abrasion which may lead to the presence of artifacts in MRI diagnostics.

## 9. Liability

It is the responsibility of the user to check the products prior to use to ensure that they are suited for the intended purpose. In case of contributory negligence by the user, Bosphorus partially or totally declines liability for all resulting damages, particularly if these are due to non-observance of our recommendations for use or warnings as well as inadvertent misuse by the user. Store products out of children's reach. For dental use only

## 10. Disposal

Resistant contamination, discard instruments in tight, puncture resistant containers.

**Any serious adverse event occurring during use and/or life cycle of the device should be reported to the manufacturer, notified body and the local component authority.**

**Dispose of as contaminated medical waste.**

## Symbols of Label:

	5.7.7 Medical device
	5.7.10 Unique device identifier
	5.1.1 Manufacturer
	5.1.6 Catalogue Number
	5.1.5 BATCH CODE
	5.1.3 Date of manufacture
	5.1.11 Country of manufacture
	5.4.3 Consult Instructions For Use
	5.4.4 CAUTION
	5.2.7 Non-Sterile
	5.3.1 Fragile, handle with care
	5.3.4 Keep dry
	5.3.2 Keep away from sunlight
	5.2.8 Do not use if package is damaged and consult instructions for use
	According to EN ISO 6360-1,2,3,5,6,7
	Defines type of operative burs

**TEK METAL SAN. VE TIC. A.S.**  
İkitelli Org.San.Bolgesi Demirciler  
San.Sit C-2 Blok No:209 İkitelli-  
Başakşehir-İstanbul – TURKEY  
Postal Code: 34490

Tel: +902126715104

Web: [www.bosphorus.com.tr](http://www.bosphorus.com.tr)

E-mail: [info@bosphorus.com.tr](mailto:info@bosphorus.com.tr)

## MILLING MODELS AND APPLICATIONS

No	Product Model	Product Images	Length Measure	Colour	Applications												
					Prophylaxis	Crowns Preparations (cutting)	Old Fill Removal (carving)	Crowns and Bridge Technique (cutting)	Model casting Technique (cutting)	Cavity Preparations (carving)	Mouth Surgery (cutting)	Acrylic Technique (carving)	Tooth Orthopedics (cutting)	Model Making (cutting)			
1	Mumalevi		Standard Short L XL	Green Black Yellow Blue Red	X	X		X	X								
2	Cylinder				X	X	X	X	X	X							
3	Topbag				X	X	X	X	X	X	X	X	X				
4	Reverse Tapered				X	X	X	X	X	X							
5	Torpedo					X											
6	Conical				X	X	X				X						
7	Needle				X	X		X	X							X	
8	Eeg				X	X	X	X	X	X							
9	Wheel					X				X							

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